AoN implants s.r.l.
Sede operativa: Via A. Manzoni, 25 - 36040 Grisignano di Zocco (VI)
Tel. +39 0444 167956 - Fax +39 0444 167957
www.aonimplants.com - info@aonimplants.com

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PROSTHETIC COMPONENTS FOR DENTAL IMPIANTS

DESCRIPTION

The AoN Implants prosthetics called Healing Caps or Abutments in the variants: Straight, Angled, Calcainable, Temporary and Customizable, are prosthetic structures, prefabricated for AoN Implants dental implants

MATERIALS

All materials used are of high quality and conform to reference standards. AoN Implants prosthetics are realised with materials supplied by qualified companies and conform with international reference standards.

All materials used guarantee an high degree of biocompatibility. AoN Implants prosthetics are sold in non-sterile packages. It is recommended, however, that the opening of packaging be done in aseptic surroundings and possibly over disinfected or sterile worktops and surfaces.

INTENDED USE

Prefabricated prosthetic structures, above named, are applicable to AoN Implants dental implants

CONTRAINDICATIONS

The use of AoN Implants prosthetics does not present any particular contraindications and it is suggested that you refer directly to the subsequent applications of dental implants and to their instructions to which it is necessary to refer for a clear indication.

Remember, however, that some contraindications are generally envisioned, such as: presence of chronic and acute infective diseases, systemic diseases, chronic sub acute maxillary osteitis, diseases involving microvascular disorders.

The use of AoN Implants prosthetics is discouraged in patients who do not cooperate, those who abuse of drugs and alcohol and those that have poor oral hygiene.

WARNINGS

The use of AoN Implants prosthetics is reserved exclusively to personnel who are familiar with prosthetic techniques for dental implants. The procedures to be employed are varied and complex; therefore, we defer to the competence of the specialist for the evaluation of the best technique to use.

The insertion of the prefabricated prosthetic structures on implants must be effected in surgically appropriate circumstances. An erroneous or improper use can cause the failure of the prosthetic operation.

INSTRUCTIONS AND MODALITY OF USE

Before insertion in the mouth, the Healing Screws and the abutments must be carefully cleaned and disinfected. On completion of use and when it is necessary to proceed with the disposal of the Healing caps and abutments, in conventional ways, repeat the cleaning and sterilisation.

The AoN Implants prosthetics are of single use; this serves to reduce the risk of potential technical damage, due to previous uses, which could compromise the work or realising the prosthetic operation.

Healing cap: on achieving osseointegration of the implant, in order to predispose the mucosa to a non-traumatically reception of the abutment, and to access the cover screw and replace it with the healing cap of a height adequate for the thickness of the mucosa, through a simple surgery.

The healing time is around 15 to 30 days but may vary according to individual cases.

abutment: upon healing of the mucosa, replace the Healing cap with a suitable abutment, fixing it to the bottom of the implant by means of the retention screw included in the pack, then secure the prosthesis on the abutment itself.

For the preparation of the prosthesis and the possible shape of the abutment, clearly it is necessary to take an imprint and execute precise work in the laboratory.

Calcainable abutment: complete the appropriate modeling working in the laboratory with the aid of the casts realised according to the imprints from the patient, and then proceed with melting and finishing of the abutment thus obtained that will be fixed to the implant by means of the retention screw which is provided.

Fix the prosthesis on the abutment. The retention screws of the abutment must be tightened with a force of less than 25N/m.

USE BY DATE

There are no special storage conditions although it is always preferable to store in a cool, dry place and not expose the product to direct sources of heat or sunlight. The disposal of the product when no longer useable must follow the existing procedures on the subject of medical waste products.

CLEANING AND STERILISATION:

The processes of cleaning and sterilisation are intended to safeguard the health of patients and of all persons working in the surgery.

Cleaning: can be manually accomplished with hot water and a suitable neutral non-corrosive detergent, using plastic or nylon brushes (never steel wool or metal brushes) to remove any organic residues.

The process of hand washing is effective if done with maximum care.

Cleaning can also be carried out in ultrasonic tanks, in a manner that is faster and safer, by fully immersing the pieces in washing liquid, and following the times recommended by the manufacturer of the equipment. Replace the washing solution according to the usage time limits recommended by the manufacturer. After ultrasonic cleaning, thoroughly rinse to remove the washing solution. Immediately after rinsing, before sterilisation, inspect and dry them.

Sterilisation: remember, first of all, that sterilisation does not replace cleaning. Sterilisation is a process that kills all microorganisms and it must be undertaken with maximum care after cleaning. The sterilisation methods most common and effective are the following:

1) Calcainable Abutments: the devices are immersed in a cold sterilizing chemical solution
2) abutments - Autoclave: the tools are treated with chemical vapours at high pressure. Times*: 20 minutes at a temperature of 130°C (270°F). Sterilising liquid: distilled water. Always follow, however, the instructions provided by the manufacturer of the equipment.

DISPOSAL

Before disposing of the used tools by conventional ways, the cleaning and sterilization of the same is required.

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